

10 THERA

2 Line HIFU



Tentech Item

10 THERA

							
10 THERA 2 Line HIFU	10 THERMA Monopolar RF	10 PL 168 Cell Fractional IPL	10 CELL 3cm x 3cm CO2 Fractional Laser	ONE THERA Dual Mode Ultrasound System	TEN HI Fractional & Thermal RF	10 SONO DSS & NBS Dual Mode System	Dr.Oracle Dr.Oracle LED Mask 4,474 LED Lights



	10 mono Thread Mono Thread [Registered for NMPA]	 REGEVAN FILL RHINO FILL FINE FILL VAGI FILL Monophasic Filler [for Export]			
	10 cog Thread Asymmetrical Cog Thread		Regevan^C Growth Factor Ointment	Regevan^S Growth Factor Spray	Regevan^A ALO CARE Growth Factor Ampoule

now **10 THERA**

- 1 Powerful lifting effect with just one treatment
- 2 V-line improvement effect
- 3 1 Line for other device “2 Line irradiation type” for 10Thera
- 4 Faster treatment than conventional methods
- 5 Powerful collagen regeneration effect






“HIFU”(High Intensity Focused Ultrasound) generates heat of
60 to 80 degrees in the dermis and SMAS layer to form thermal coagulation point
And regenerates collagen fiber and elastic fiber for lifting effect

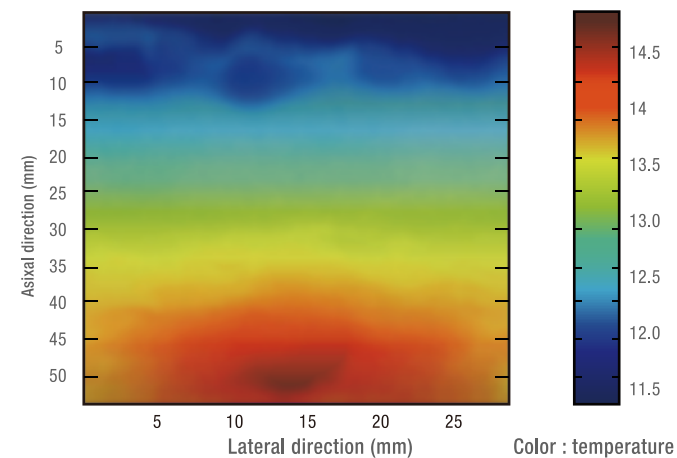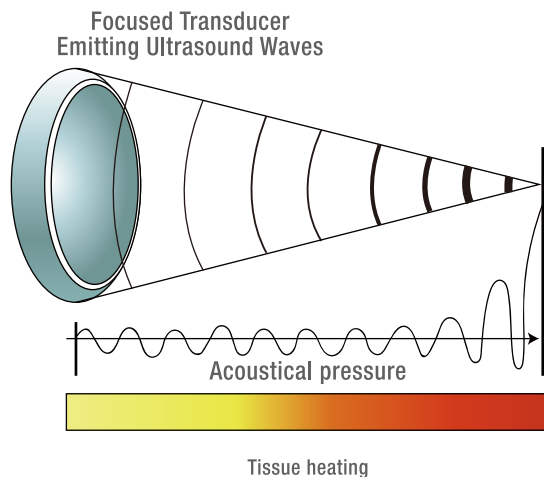


HIFU Comparison

10 THERA

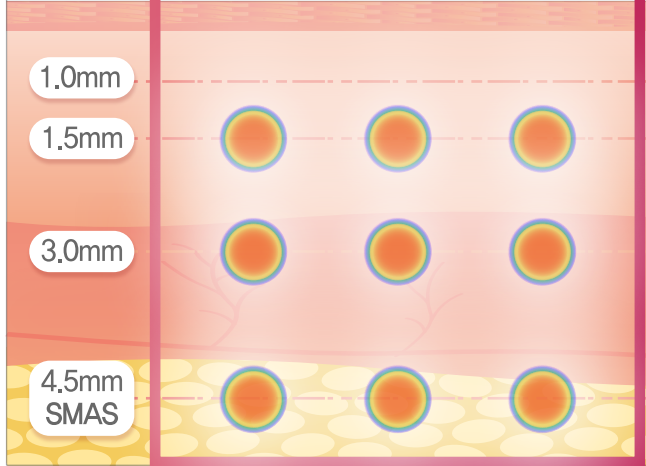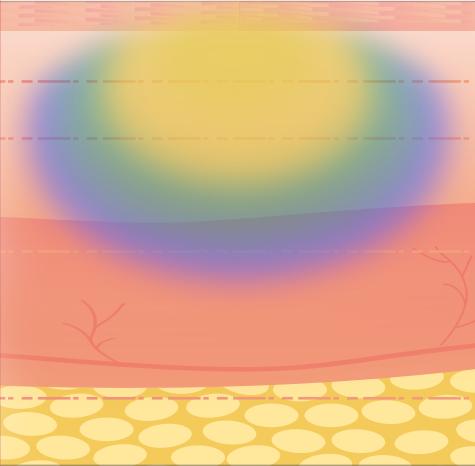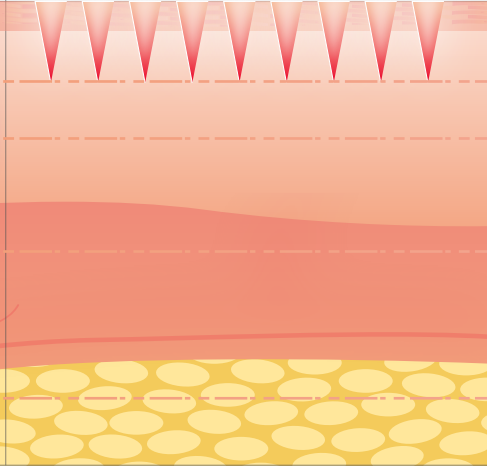
	HIFU	RF	Laser
Temperature	60 - 80°C Coagulation /Denaturation	< 55°C Bulk Heating	50 - 100°C Vaporization / Ablation
Depth	4.5 mm, 3.0 mm	< 3 mm; Variable	< 1.5mm; Superficial
Skin surface	Perfectly Preserved	Heated	Ablated
Features	Precise & Fractional	Bulk Heating	Variable
1.0mm			
1.5mm			
3.0mm			
4.5mm SMAS			

2 Line Type HIFU Device

10THERA 1 Shot

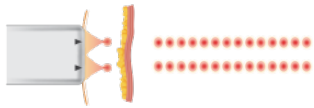

other 1 Shot

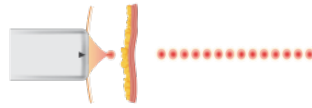

Improves the overall face by raising the temperature of the SMAS layer and the dermal layer to 60 ~ 80 degrees and inducing the regeneration of collagen.



2 Line HIFU



Cartridge	 4Mhz 4.5mm	 7Mhz 3.0mm
Type	Hifu	Hifu
Power	0.1~1.5J (0.05 Step)	0.1~0.5J (0.05 Step)
Length	5.0~25mm (1.0 Step)	5.0~25mm (1.0 Step)
Spacing	1.5, 1.8, 2.0mm	1.5, 1.8, 2.0mm

SPEC

10THERA

User interface	15inch 24bit Touch LCD
Power	AC 100~230V, 50/60HZ
Dimension	W 550 x D 520 x H 1200mm
Weight	38kg
Electrical Requirement	100-240V~, 50/60Hz, 300VA



10THERA Advantages

10THERA




Confident lifting with 2 Line HIFU!



10THERA 200 Shots,

Similar effect as 400 shots in other HIFU

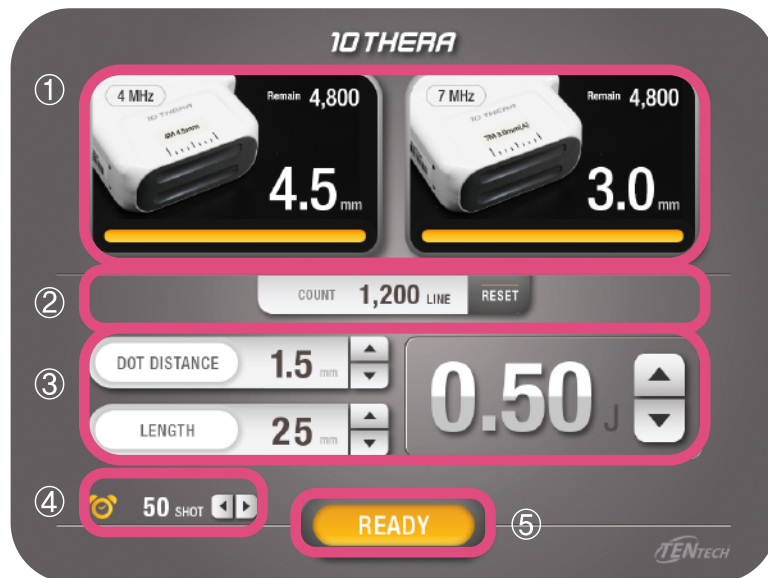

- ① Recognized Cartridge: Displays recognized cartridge information
- ② Cartridge Info: Number of Shots (Lines) Used / Reset Button
- ③ Parameter value adjustment
: Power (Output control) / Spacing (Irradiation interval) / Length (Irradiation distance)
- ④ Standby / Ready
- ⑤ Alarm : 50 Shots (100 Lines) Alarm can be set in units

Before & After

10 THERA




Treatment Info

- Age : Female in late 20s
- Parameter : 4Mhz 4.5mm 0.7J 100shot / 7Mhz 3.0mm 0.4J 30shot
- Time elapsed : 3 weeks
- Results : Volume on the front of the face looks lively due to the increase in the elasticity of the cheekbones

Before & After

10 THERA

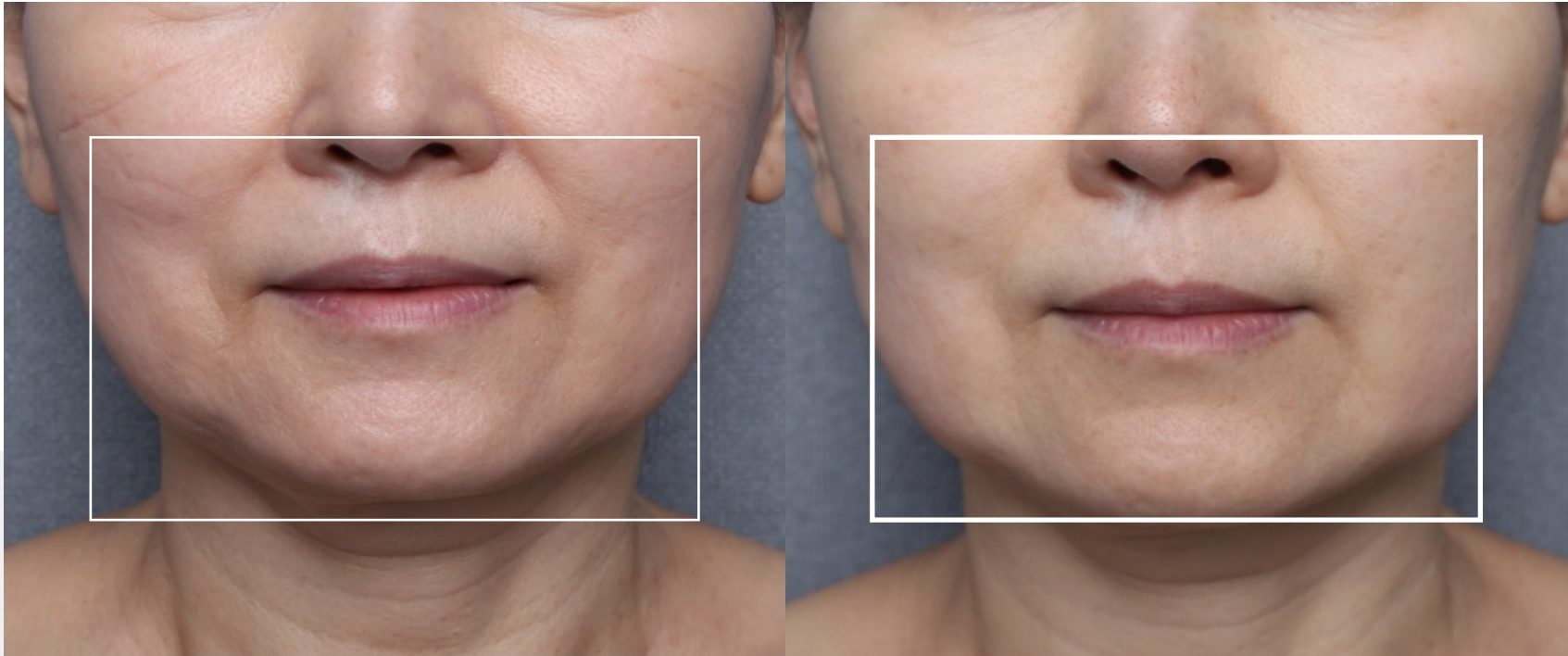


Treatment Info

- Age : Female in early 60s
- Parameter : 4Mhz 4.5mm 0.5J 60shot / 7Mhz 3.0mm 0.3J 60shot
- Time elapsed :1 week
- Results : Overall skin tone is brightened, fine lines around the mouth are improved, and elasticity is formed

Before & After

10 THERA

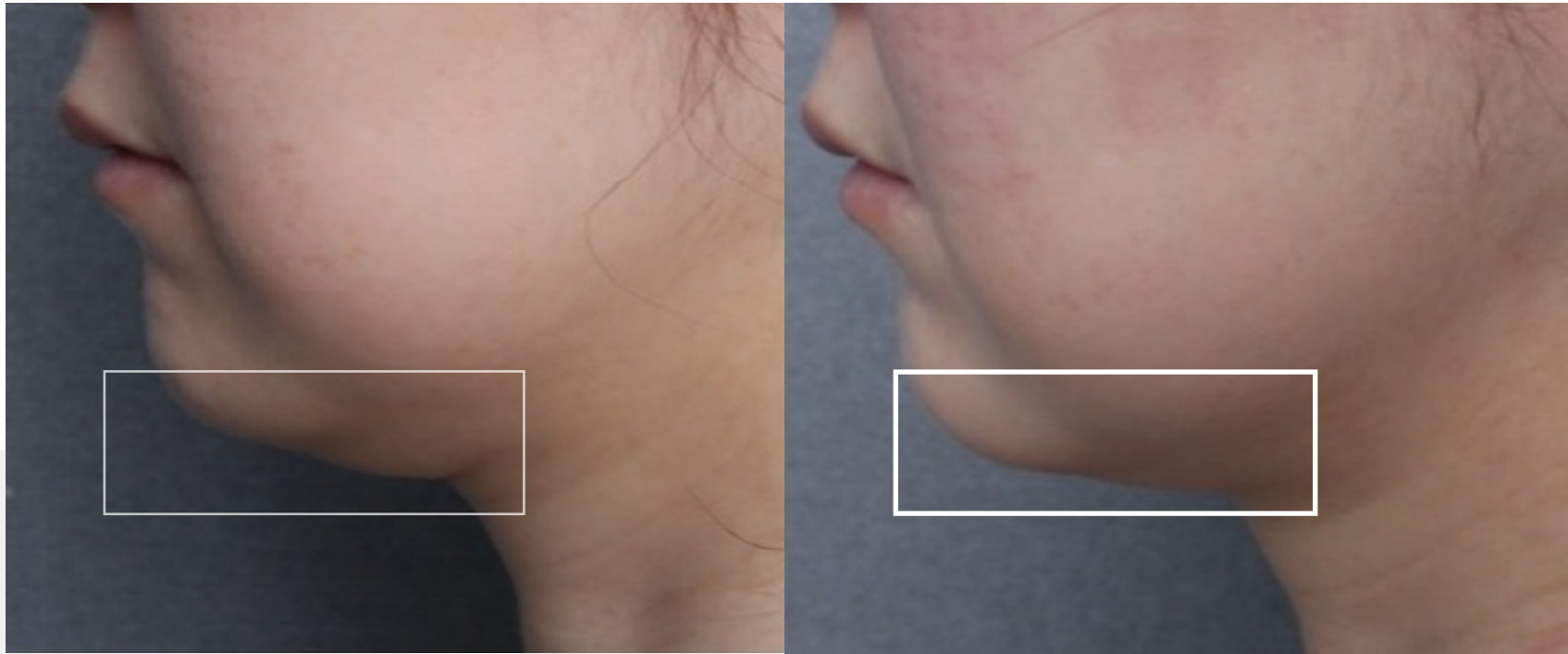

Treatment Info

- Age : Female in early 60s
- Parameter : 4Mhz 4.5mm 0.5~0.6J 50shot / 7Mhz 3.0mm 0.3J 15shot
- Time elapsed : On the day of treatment
- Results : Worried about the collapsed face line, expecting effect over time

Before & After

10 THERA



Treatment Info

- Age : Female in mid 30s
- Parameter : 4Mhz 4.5mm 0.5J 60shot / 7Mhz 3.0mm 0.3J 60shot
- Time elapsed : 1 week
- Results : Overall skin tone is brightened, fine lines around the mouth are improved, and elasticity is formed.

Before & After

10 THERA



Treatment Info

- Age : Female in mid 30s
- Parameter : 4Mhz 4.5mm 0.5J 60shot / 7Mhz 3.0mm 0.3J 60shot
- Time elapsed : 1 week
- Results : Overall skin tone is brightened, fine lines around the mouth are improved, and elasticity is formed.

10 THERA

All information in this document is subject to copyrights and other intellectual property rights of Tentech Inc. and other licensees. No modification, reproduction or copy of any or any part of this material may be made without prior written permission from Tentech Inc. and licensees. Additional information is available upon request. The following are trademarks or registered trademarks of Tentech, Inc. in Korea and other countries.



10THERA, fast and efficient
2 Line Type HIFU Device



Epidermis

Dermis

Subcutaneous Tissue

SMAS